
Air Sport™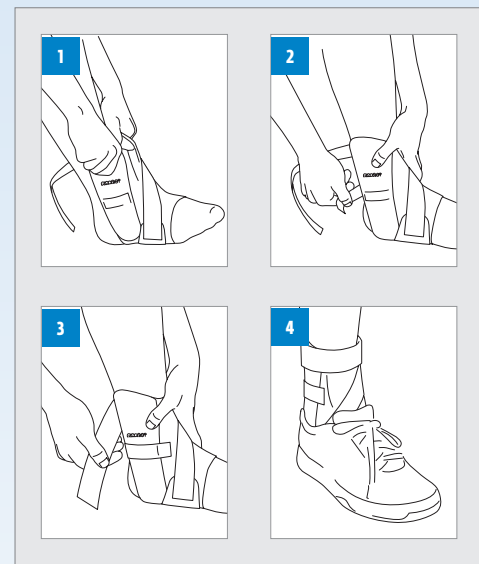

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The Aircast Airsport Ankle Brace is designed to provide support and protection for healthy ankles during sport activities. For maximum comfort wear an absorbent sock and a laced athletic shoe when wearing the brace. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS:

N/A

WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
 - Do not use over open wounds.
 - Do not use this device if it was damaged and/or package has been opened.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- 1) Unfasten straps and open brace. Slide foot into back of brace. Pull brace shells upward and align shells with sides of ankle. (Fig. 1)
- 2) Wrap Achilles strap around back of ankle and secure snugly. (Fig. 2)
- 3) Pull the ATF cross-strap until it is snug. Wrap around back of ankle and secure snugly. (Fig. 3)
- 4) Put on lace shoe or sneaker. Note: To adjust fit, tighten or loosen ATF cross-strap. (Fig. 4)

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

 **FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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